SIMPLIFY

NEEDLE HOLDER
for image guided interventions
Instructions for use

Important: Federal (USA) law restricts this device to sale by or on the order of a physician. Store in a dry, cool, dark place. Sterile, for single use only.

Content

Simplify™ needle holder – Instructions for use (GB)	4
Simplify™ nåleholder – Bruksanvisning (NO)	8
Simplify™ nålhållare – Bruksanvisning (SE)	10
Simplify™ naaldhouder – Gebruiksaanwijzingen (NL)	12
Porte-aiguille Simplify™ – Mode d’emploi (FR)	14
Simplify™ nåleholder – Brugsanvisning (DK)	16
Simplify™ Nadelhalter – Gebrauchsanleitung (DE)	18
Simplify™ porta aghi – Istruzioni per l’uso (IT)	20
Portaagujas Simplify™ – Instrucciones de uso (ES)	22
Porta-agulhas Simplify™ – Instruções de utilização (PT)	24

Simplify™ needle holder – Instructions for use (GB)

General use information

The product is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques should be employed.

Packaging

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use only. Sterile if package is unopened or undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light.

Product description

Attached to Simplify the needle will be kept in a chosen angle even if the needle is at a superficial depth and not sufficiently supported by the tissue. When acquiring new images the user will get a firm idea of the planned direction of the needle and can redirect the needle until the correct angle is obtained without having to insert the needle further. The needle holder prevents the needle from changing its preselected angle due to gravity.

There are three different clamp sizes marked according to the corresponding needle size; 14/15G, 17/18G and 20/21G.

The orientation of the clamp is correct when the slit opening is towards the Simplify logo.

When the clamp is in the upper groove it shall move freely along the arc. When pressed down to the second groove it shall lock in position.

Indications for use

Simplify is a detachable needle holder to be used for image guided percutaneous procedures.

Restricted use

Needles above 20 grams and longer than 30 cm.


Warnings and precautions

The needle holder should be loosened after puncture so as not to restrict unnecessarily the body's normal internal movements.

Preparation

Remove all but the appropriate clamp (figure 1) matching the chosen needle size.


Figure 1: Remove all but the one clamp matching the selected needle size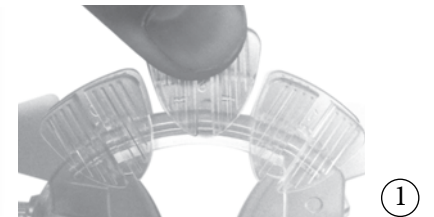

Instruction for use


1. Remove the release liner and place the needle holder centred on the entry point (figure 2)

Figure 2: Place the needle holder centred on the entry point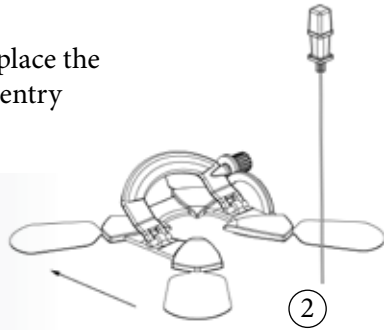

2. Remove the chosen clamp, fit it around the needle (figure 3) and press it lightly onto the arc until a click can be heard


Figure 3: Fit the clamp around the needle and press down for the first click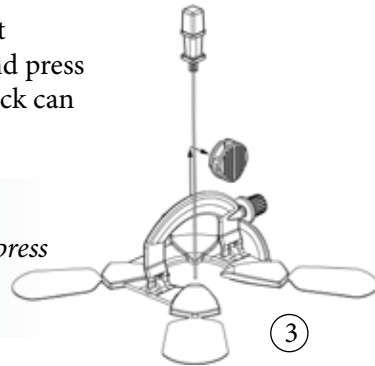

3. Move the assembly until the appropriate angle is obtained

4. Fasten the nut and push down the clamp (figure 4) to fixate the needle in the chosen angulations

Figure 4: Fasten nut and push down clamp to fixate the needle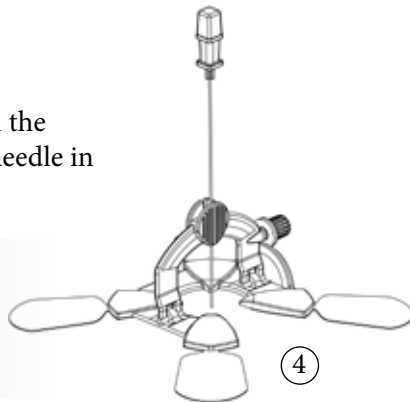


5. If diagnostic images show a need to re-angle the needle repeat 3 and 4 until the angle to the lesion is correct

6. The needle holder may be used to steer the needle towards the lesion

7. If needed remove the clamp and fold down (figure 5) the arch to continue without aid from the Simplify

Figure 5: Remove clamp and fold down arch

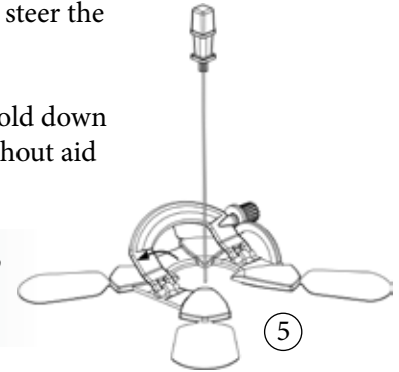Troubleshooting

If the clamp does not lock when in the lower position try removing the clamp and pressing the lower ends together for a tighter fit.

References

These instructions for use are based on experience from physicians. As more data become available other factors may become evident. Refer to your local sales representative or NeoRad for information on available literature.

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